



DENTAL INSTANT CAMERA

OPERATION MANUAL



PREFACE

We much appreciate your kindly selection for our portable dental X-ray unit, please read the operation manual carefully before using, so that you could learn all the details about technical specification, installation, operating methods and maintenance.

Please connect with the dealer if you have any problem after reading the operation manual.

Thank you again.

Equipment Maintenance

Because the equipment include the users operation and diagnosing patients, therefore must be maintained, checked, diagnosed equipment maintenance every year, make sure the equipment safety. And this equipment involved in high voltage electrical control components, should check the safety of its insulation, grounding device reliability.

CONTENTS

ABOUT THE SYMBOL	3
INTRODUCTION	4
SPECIFICATIONS	6
STRUCTURE	7
OPERATION	8
WARNING AND NOTICE	12
MAINTENANCE AND KEEPING	14
PACKING LIST	14

Chapter 1 ABOUT THE SYMBOL

Serial number	Symbol	Explain
1		High voltage
2		X-ray emitting
3		Caution
4		Power off
5		Power on
6		Ground wire connector
7		Fragile
8		Up sign
9		Max accumulation
10		Dryness
11		Dose permission sign
12		BF type

Chapter 2 INTRODUCTION

2.1 Characteristics and main use

This portable dental X-ray unit is high frequency machine. The body is small, light weight and nearly no radiation. It has excellent image quality, portable storage, save more space. It uses high frequency and DC international power supply. All of the components installed in the central PC board concentrated. Shock, set-up, electron tubes, all of them are insulation vacuum, sealed stereotype protection. There are also manual buttons installed on the surface of shell, as well as batteries and charges. This unit mainly suitable for oral pre-treatment for learning internal organizational structure, root canal therapy, dental implant and so on, is indispensable in daily life clinic equipment, especially for dental implant surgery. It also can connect with sensor, is great convenience.

2.2 Checking and Security

Caution: Please learn relative security measures and functions given in the file or on the unit.

- Operating the device in the designated order.
- Keep the device and the cone clean.
- Please use or operate the device and parts after taking a necessary periodic inspection.
- If you discover some difference in the sensor images even if you inspected its x-ray exposure time in the same way as before, then it means that the radiation of x-ray is abnormal, you must stop using it immediately and allow it to be inspected.
- Keep the device in the place where children are unable to touch.
- Don't use or store near the place which

- is possible to fire.
- Don't use or store where air pressure or temperature or moisture exceeds range.
- Keep good ventilation and avoid direct sunlight, protect against the eroding of dust and corrosive air.
- Don't use or store near to chemicals or combustible gas.

Chapter 3 SPECIFICATIONS


3.1 X-RAY CAMERA

Tube voltage	60 (70) kV	Tube current	2.0 (1.0, 1.5, 2.5) mA
Exposure time	0.1~2S	Frequency	30kHz
Tube focus	0.8*0.8 (0.4*0.4) mm	Rated power	120 VA
skin to cone	130mm	Leak radiation	0.005mR
Battery	DC14.8V, 1300mA	Charger	Input: AC100V-240V Output: DC16.8V
Image display	4.0 inch touch screen	Resolution ratio	400*800
REM	2G	ROM	16G
Bluetooth	5.0 TWO-WAY	WiFi	2.5G/5G Dual frequency
USB	2.0 OTG	Battery	DC-3.7V, 1600mA
Net weight	1.6 kg	Machine size	220*125*150 mm
Gross weight	3.0 kg	Package size	300*250*200 mm

3.1 X-RAY CAMERA

MODEL	COX-2230	COX-2635	μm
Technology	CsI	CsI	pixels
Effective area size	22.2*29.6mm	26*35 mm	mm
Dimensions	27.8*38.9*5mm	32.3*44.3*5mm	bit
Digitalize	16bit	16bit	lp/mm
Resolution ratio	20 lp/mm	20 lp/mm	
Output *	USB2.0	USB 2.0	dB
Cable Length	2m(Extendable)	2m(Extendable)	e-/sec
Service temperature	0~40	0~40	ke
Water proof	IP68	IP68	kRad
Support System	Win 7,8,10 and fit	Win 7,8,10 and fit	


Chapter 4 STRUCTURE

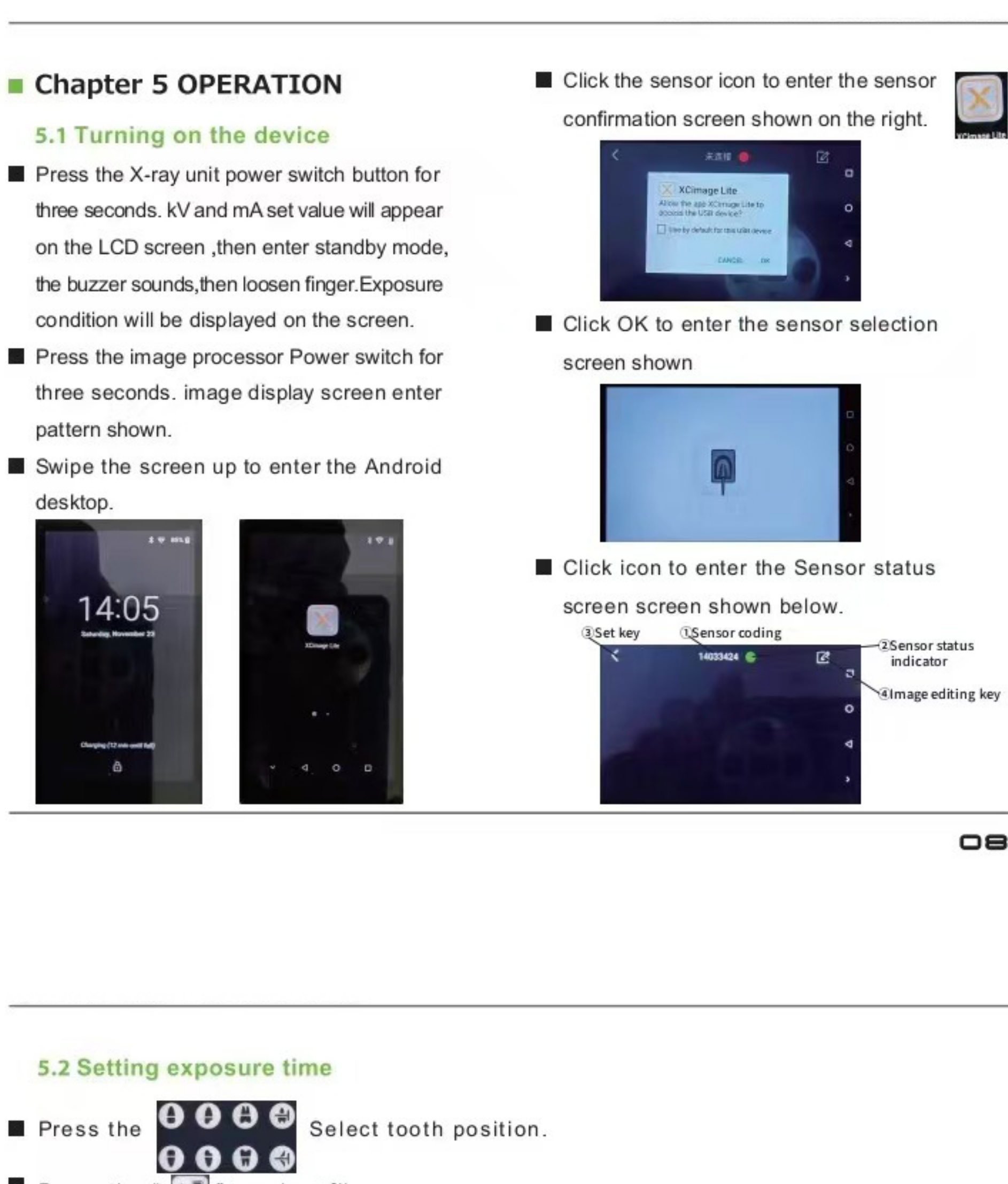


Chapter 5 OPERATION

5.1 Turning on the device

- Press the X-ray unit power switch button for three seconds. kV and mAs value will appear on the LCD screen, then enter standby mode, the buzzer sounds, then loosen finger. Exposure condition will be displayed on the screen.
- Press the image processor Power switch for three seconds. Image display screen enter pattern shown.
- Swipe the screen up to enter the Android desktop.

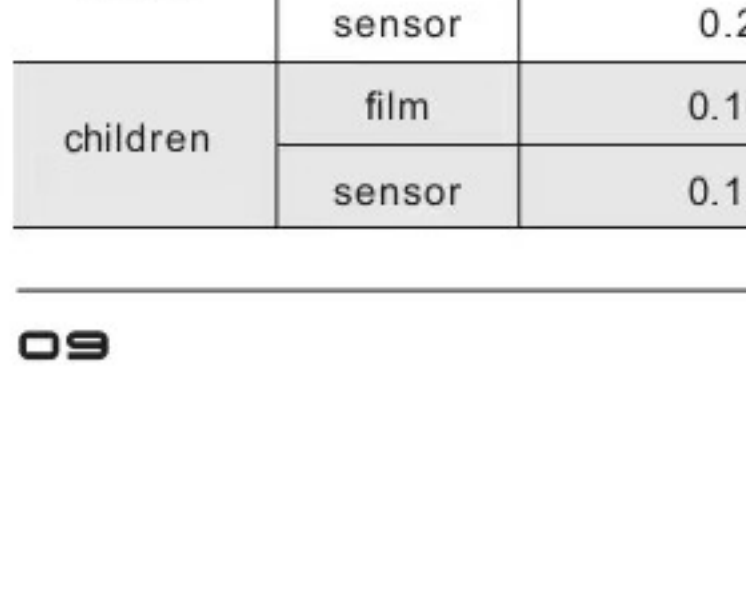


- Click the sensor icon to enter the sensor confirmation screen shown below.

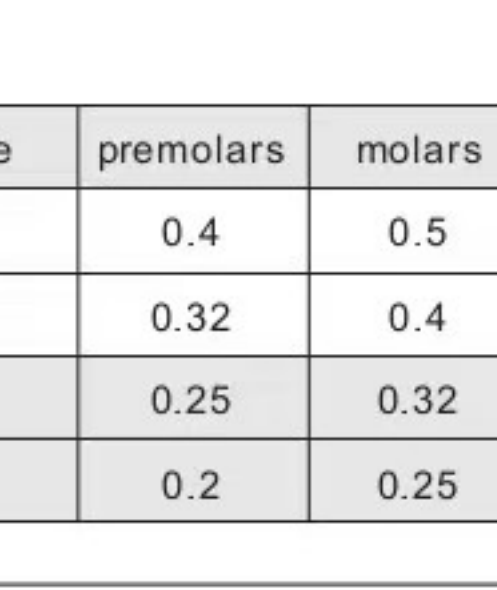

- Click OK to enter the sensor selection screen shown.



- Click icon to enter the Sensor status screen shown below.

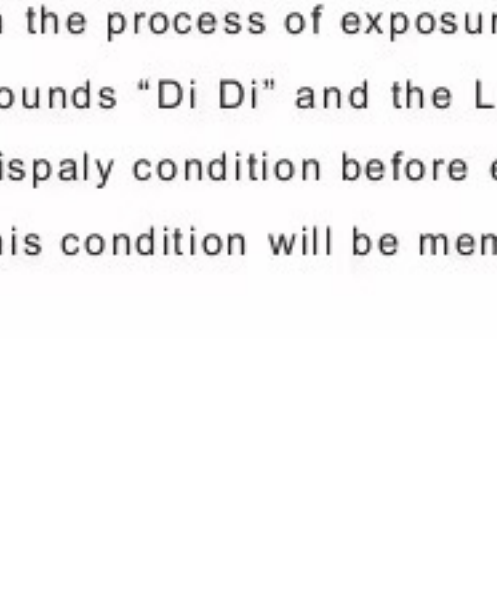

5.2 Setting exposure time

- Press the Select tooth position.
- Press the to select film or sensor.
- Press the to select adult or child.
- Press the or to increase or reduce the exposure time for each step if necessary.
- The device will be turned off automatically if there is no any operation for 5 minutes.

5.3 Exposure time

	tooth position	central incisors	canine	premolars	molars	occlusal
adult	film	0.25	0.32	0.4	0.5	0.63
	sensor	0.2	0.25	0.32	0.4	0.5
children	film	0.16	0.2	0.25	0.32	0.4
	sensor	0.13	0.16	0.2	0.25	0.32

5.4 Place the film(or sensor)

- Put the sensor or a film in the mouth, keep film upright when select front tooth, horizontal when select back tooth, make the beam cone vertical to film or sensor.

5.5 Exposure

- Press hold the exposure button (until the end of the exposure), x-ray will be exposed as the setting time and blue LCD is display "EXP" and the exposure LED is light.
- In the process of exposure, the buzzer sounds "Di Di" and the LCD turned to display condition before exposure and this condition will be memorized.


5.6 Turning off the device

- Press the power switch button for two seconds and then loosen finger, the device will be turned off.

5.7 Charging battery

- High efficiency lithium battery enables you to irradiate x-ray more than 120 times after charging one time, and the battery are not easily to changed even for long-term use.
- Using only the authorized charger provided as an accessory when charging the battery.
- The image processor charger output: 5V ± 1A.
- The X-ray unit charger output: 16.8V ± 1A.

as shown in the diagram

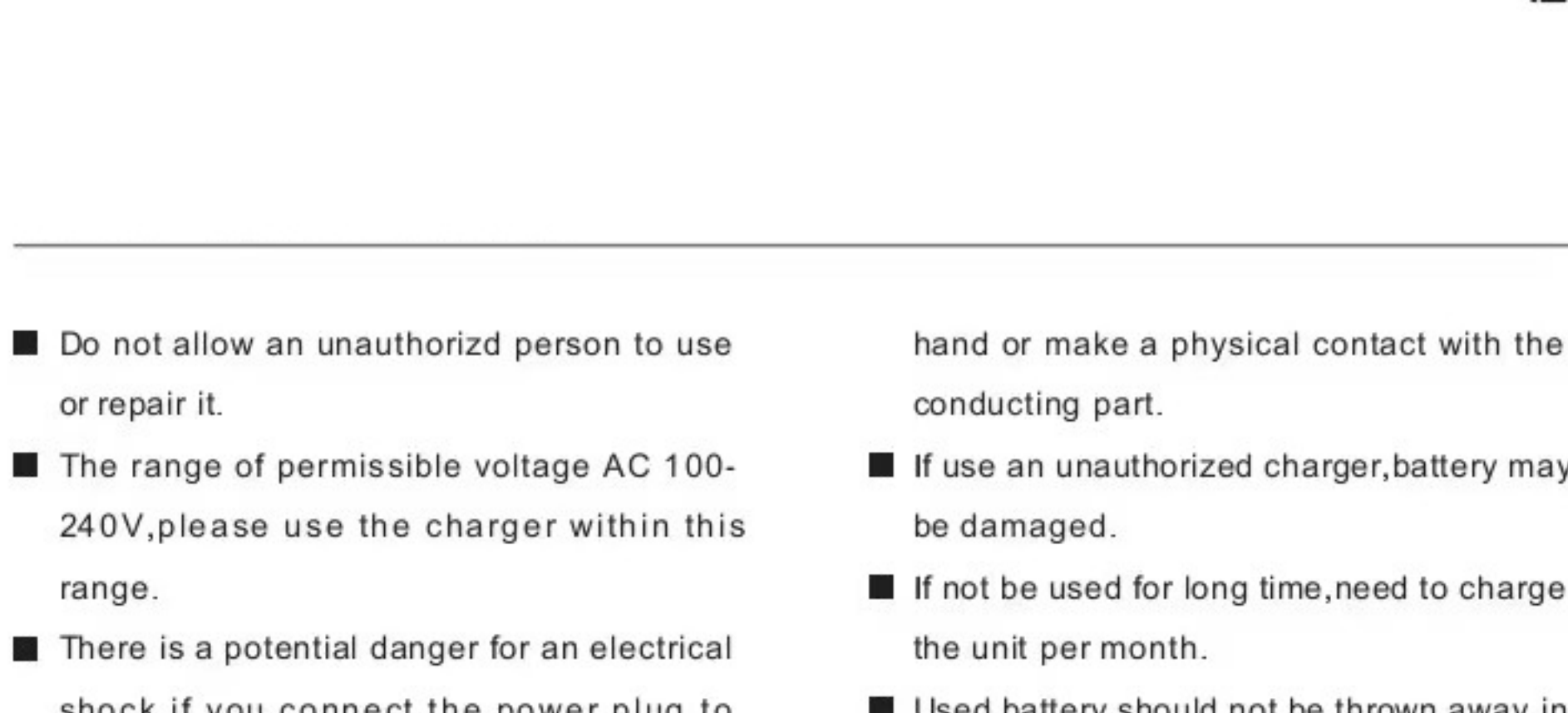

Chapter 6 WARNING AND NOTICE

6.1 Warning

- Warning About Using:
 - Make sure the person using this device is legally.
 - If pregnant, ask a health professional before taking x-ray.
 - An excessive exposure of radiation leaking may cause human body reaction lightly.
 - Humidity conditions recommended for storage: 10-85%RH.
 - Humidity conditions recommended for use: 15-80%RH.
 - Optimal humidity range: 15-60%RH.
 - Temperature conditions recommended for storage: 10-50°C.
 - Temperature conditions recommended for use: 10-35°C.
 - Optimal temperature range: 10-25°C.

Other Warning:

- If any problem or error founded in the device, stop operating and then consult with the designated A/S center.
- Do not disassemble or modify the device, since it includes generates x-rays, if not use properly, it may damage users or patients.

Chapter 7 MAINTENANCE AND KEEPING

- Keep the device in the place where only authorized personal can reach.
- Please clean the device daily by cleaning clothes and must keep cone clean.
- Please do not remodel the device arbitrary.
- Recommend to check the device once a year at the designated A/S center for safe use.
- Please recharge the battery every three months when it not in use for long time and keep it in store.
- Daily tests and annual maintenance.
- When turn on, please confirm error message is not display on the LCD screen.
- Please confirm whether buzzer sound is heard and "EXP" words on the LCD, and exposure LED could see normally at x-ray exposure.
- Every year for safe use, safety check-up is necessary.

Chapter 8 PACKING LIST

No.	Name	Piece
1	Main body	1
2	Sensor	1
3	X-ray unit Charger	1
4	U disk	1
5	Image display charging cable	1
6	Sensor conversion line	1
7	User manual	1
8	Battery maintenance instructions	1
9	Important Reminder	1

Chapter 9

Chapter 10

Chapter 11

Chapter 12

Chapter 13

Chapter 14

Chapter 15

Chapter 16

Chapter 17

Chapter 18

Chapter 19

Chapter 20

Chapter 21

Chapter 22

Chapter 23

Chapter 24

Chapter 25

Chapter 26

Chapter 27

Chapter 28

Chapter 29

Chapter 30

Chapter 31

Chapter 32

Chapter 33

Chapter 34

Chapter 35

Chapter 36

Chapter 37

Chapter 38

Chapter 39

Chapter 40

Chapter 41

Chapter 42

Chapter 43

Chapter 44

Chapter 45

Chapter 46

Chapter 47

Chapter 48

Chapter 49

Chapter 50

Chapter 51

Chapter 52

Chapter 53

Chapter 54

Chapter 55

Chapter 56

Chapter 57

Chapter 58

Chapter 59

Chapter 60

Chapter 61

Chapter 62

Chapter 63

Chapter 64

Chapter 65

Chapter 66

Chapter 67

Chapter 68

Chapter 69

Chapter 70

Chapter 71

Chapter 72

Chapter 73

Chapter 74

Chapter 75

Chapter 76

Chapter 77

Chapter 78

Chapter 79

Chapter 80

Chapter 81

Chapter 82

Chapter 83

Chapter 84

Chapter 85

Chapter 86

Chapter 87

Chapter 88

Chapter 89

Chapter 90

Chapter 91

Chapter 92

Chapter 93

Chapter 94

Chapter 95

Chapter 96

Chapter 97

Chapter 98

Chapter 99

Chapter 100

Chapter 101

Chapter 102

Chapter 103

Chapter 104

Chapter 105

Chapter 106

Chapter 107

Chapter 108

Chapter 109

Chapter 110

Chapter 111

Chapter 112

Chapter 113

Chapter 114

Chapter 115

Chapter 116

Chapter 117

Chapter 118

Chapter 119

Chapter 120

Chapter 121

Chapter 122

Chapter 123

Chapter 124

Chapter 125

Chapter 126

Chapter 127

Chapter 128

Chapter 129

Chapter 130

Chapter 131

Chapter 132

Chapter 133

Chapter 134

Chapter 135

Chapter 136

Chapter 137

Chapter 138

Chapter 139

Chapter 140

Chapter 141

Chapter 142

Chapter 143

Chapter 144

Chapter 145

Chapter 146

Chapter 147

Chapter 148

Chapter 149

Chapter 150

Chapter 151

Chapter 152

Chapter 153

Chapter 154

Chapter 155

Chapter 156

Chapter 157

Chapter 158

Chapter 159

Chapter 160